


HeroShield Instructions

Product:	HeroShield	Manufacturer:
Model No.(s):	FS-001-A, FS-001-B - Colours may vary	Denroy Plastics LTD, 9-11 Balloo Drive, Balloo Industrial Estate, Bangor, Co. Down, BT19 7QY
Classification:	PPE Class I of REGULATION (EU) 2016/425	www.denroy.com


Assembly

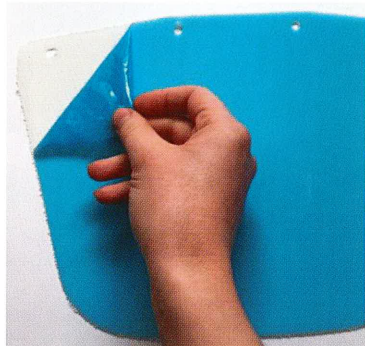

1) Peel away any film attached to the shield component. Film may be on both sides of the shield material.



2) Attach the shield to the visor band working from one outside corner to the last corner and securing all 4 holes in place on the off-stands of the visor brim

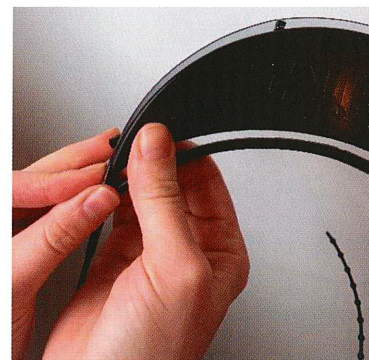

3) Feed the ball chain through the key-hole from above and adjust to fit comfortably - do not overtighten (for your comfort)

Terms of Supply

The Recipient acknowledges and agrees that the Goods are supplied on an "as-is" basis, and while the Goods have been manufactured using reasonable care and skill, to the maximum extent permitted by applicable law, the Suppliers hereby exclude from these Conditions all warranties in respect of the Goods, express or implied, including but not limited to sections 13 to 15 of the Sale of Goods Act 1979. Full terms of supply can be found at www.heroshield.net/pages/terms-of-supply

Use and Care Instructions

The HeroShield is intended to be worn by Health Care Professionals to protect the face from liquid splashes and bodily fluids when treating infectious patients. It is not designed to prevent acute respiratory infections (ARIs) or to protect from impacts. It is highly recommended that eye protection and a FFP3 Respirator are used with the Hero Face Shield when appropriate to protect against respiratory infections or diseases which may be present. The user should follow all safety protocols as recommended by their specific organisation during all **Aerosol Generating Procedures (AGP)**.

The visor can be used as short-term wear of up to 8 hours. A reused headband should be used by the same person only. Do not share or reuse headbands between different people under any circumstances.

Maintenance and Servicing:

low risk use

HeroShield can be disinfected with alcohol wipes or isopropyl alcohol. The headband component can be disinfected with boiling water if other methods are unavailable.

For NHS / high risk use

The band and visor may be sterilised for reuse by appropriate hospital (low temperature) procedures. Alternatively, HeroShield can be incinerated/ disposed with medical waste.

For storage at room temperature

Complies with Essential Requirements, Annex II to REGULATION (EU) 2016/425
under the sole responsibility of, HeroShield Limited

Document Reference: FS-IA-01